

the
lullaby
trust

coni

Movement Monitors

CONI coordinator information



coni@lullabytrust.org.uk

Please note that families must be signposted to the manufacturer's instructions for the monitor they are using – this is a guide for you, as CONI coordinator only.

What are movement monitors?

Movement monitors sound an alarm after a pre-set time (usually 20 seconds) if they fail to detect a baby's movement. In this context they are used to detect the movement of breathing. All babies have irregular breathing patterns compared to adults, with frequent short pauses between breaths. However, a 20-second pause (or apnoea) is unusually long. Some monitors will alarm if the respiratory rate falls below a pre-set minimum.



Can movement monitors prevent sudden infant death?

Despite the widespread use by parents there is no research evidence that monitors prevent sudden infant death syndrome (SIDS) ^(2, 3). Babies are known to have died whilst on a monitor ⁽⁴⁾. Large scale studies looking specifically at the effect of monitor use on infant mortality rates have not been carried out. Since SIDS is relatively rare, such studies would have to involve an enormous number of babies to obtain reliable statistical information comparing babies on monitors to others at equal risk. The Confidential Enquiry into Stillbirths and Deaths in Infancy, the largest ever SIDS study conducted in the UK, found that monitors have a 'lack of any apparent value from such devices in the prevention of deaths.' ⁽¹⁾ In advice published in 2016, The American Academy of Pediatrics concludes monitors should not be used as strategy to reduce the risks of SIDS ⁽²⁾.



Advantages of movement monitors

Many parents who have previously had a baby die as a SIDS gain reassurance using a monitor with their next baby. They know that they will be alerted should the baby stop making (breathing) movements.

Parents say they are more able to sleep themselves knowing that the monitor will wake them if their baby stops (breathing) moving. Some parents bereaved by SIDS have said that if a monitor does nothing else, at least it gives them the chance to be with their baby if they stop breathing even if they cannot be resuscitated ⁽⁵⁾. Of parents using monitors provided by CONI, 91% have said the monitor was helpful or very helpful to them ⁽⁶⁾. A monitor can help parents to look after their baby with more confidence. Having a monitor can also help parents to follow safer sleep advice, for example by making sure that the baby sleeps on their back.

A monitor can never replace a parent or carers supervision of a baby, and if parents are ever worried about their baby's health, then they should seek medical advice.



Disadvantages of movement monitors

Whilst the use of movement monitors may give considerable psychological support to some parents, there are also disadvantages:

- Movement monitors do not detect obstructive apnoea although some may alarm if movements drop below a pre-set rate per minute.
- Movement monitors can give false alarms which may heighten parents' anxiety and lead to disturbance of the baby's sleep. False alarms may occur for a variety of reasons - the sensor pad becomes detached, or the abdominal movements become so slight that they are not detected ⁽⁷⁾.
- There is a risk of babies becoming entangled with sensor leads ⁽⁸⁾.
- Parents may rely upon the monitor for assurance that the baby is well and may not look out for other signs of illness or observe the baby's overall health and development ⁽⁹⁾.
- When cessation of breathing is detected, it may not be possible to resuscitate the baby.

There are three main types of monitors available as certified medical devices:

1. A small sensor pad held in contact with the baby's abdomen by tape and connected by wire or tubing to the monitor e.g. SISS Baby control; Graseby MR10.
2. A large sensor pad placed in the baby's cot, usually designed to be put on the base and under the mattress e.g. Nanny BM-02; Angelcare AC25-1.
3. 'Clip on'- This is a small plastic monitor that clips onto a nappy e.g. Snuza Hero^{MD}.



Other types of monitors

Respiration and oxygenation can be monitored by pulse oximeters, impedance monitors, and cardio-respiratory monitors. These are more complicated to use than movement monitors. There is no evidence that they can prevent SIDS⁽²⁾. In addition, there are many makes of movement monitors sold as nursery equipment to parents as 'reassurance' monitors but are not certified as medical devices.

Responsibilities of CONI local areas/teams

Every area should maintain their own local stock of monitors and mattress pads to offer to CONI families. The choice of monitor(s) to provide is decided at local level, and should be purchased directly from retailers/distributors, not via The Lullaby Trust. The Lullaby Trust may on occasion be able to donate monitors to local areas/teams. However, we are solely dependent upon donations from other charities to buy monitors and these are sporadic. So, our donations are intended to supplement local stocks and aren't to be relied upon to deliver the programme at local level.

Information regarding the purchase of monitors directly from retailers and distributors can be found in the CONI Guidelines for Professionals document: lullabytrust.org.uk/care-of-next-infant

We would recommend that families are always provided with a movement monitor that is classed as a 'medical device'. It can be identified if a product is a medical device with the label 'MD' or 'medical device' on the manufacturer information and/or packaging. All monitors should have a CE or UKCA mark if they are available for sale in the UK.



Please signpost families to our expert information on baby monitors: lullabytrust.org.uk/baby-monitors

The Lullaby Trust recommends that when a baby is monitored:

- Parents are shown how to use and care for the monitor supplied to them and be issued with the instruction booklet/manual and the manufacturers /distributor website to access videos and Q&A section.
- Parents are informed about the limitations of the monitor.
- Parents are supported to have a plan if the monitor alarms.
- Parents are trained in resuscitation techniques e.g. basic life support (BLS):



» The gold standard is for the health visitor/practitioner to train parents in basic life support using a resuscitation doll. However, this will be a local level protocol and not always possible. It could also be achieved in other ways. For example, the local resuscitation officer could provide training to the parents,

» Or the health visitor/practitioner could watch a video with the parents explaining basic life support. Organisations such as the British Red Cross and St John Ambulance have made high-quality videos available online on YouTube or on their websites e.g.

- [How to do CPR on a baby \(under one year old\) - Baby First Aid | St John Ambulance \(sja.org.uk\)](#)

- [How to do a baby primary survey | St John Ambulance \(sja.org.uk\)](#)



- Skin sensor-type monitors are not used after six months of age or when the baby can roll, reach and grasp as there is an increased risk of accidents with wires/tubes with an active baby.
- Parents should be advised to use the monitor for sleep periods only, in a Moses basket, cot or crib.
- Care is taken that external movement does not interfere with the monitor e.g. movement of car or pram/buggy, or on a surface which may pick up vibrations, including walking around the cot or crib on certain flooring e.g. parquet or floating floors, or sharing a sleep space with another person or animal e.g. an adult co-sleeping with their baby in bed.



- Parents and carers should be supported to continue to follow safer sleep advice.
- The period of monitoring should be discussed an estimated date for discontinuation agreed. The risk of sudden infant death declines with age and 85% of sudden deaths occur in babies under six months old, so for most babies, monitoring should stop by six months. The monitor can help reassure parents that their baby has not come to harm and many parents say the monitor helps them to sleep better at night, so the withdrawal of the monitor should be gradual. Parents should be encouraged to use it for fewer hours each day and the health visitor should help them plan how to progress until they can manage without the monitor.

Plan if monitor alarms

It is essential that parents have a plan to follow if the monitor alarms. Parents should be advised to make sure that anyone who plans to care for their baby knows what to do e.g. grandparents, babysitters etc.

The following is a summary of the measures that should be considered:

1. Switch off the alarm and check baby:

- Look to see if the baby's colour is paler than usual or blue
- Look down for movement of the baby's chest and/or tummy
- Place your cheek next to the baby's face to check whether you can feel or hear any breathing
- Stimulate the baby by talking, blowing on or touching its face, tickling the soles of the feet. Do not shake the baby

2. If one of the above points applies, shout for help and carry out the basic life support techniques that you have learned. Seek medical assistance by ringing 999 as soon as possible.

Parents should be advised that if there is no response and they are not able to carry out BLS, then they should call for help and take the baby to the phone - dial 999 and preferably put the phone on speaker. They will then be talked through BLS until professional help arrives.

3. If the baby does respond but they are worried about their health, seek medical advice as soon as possible.

More information is available from the Resuscitation Council (UK):
<https://www.resus.org.uk/>



Common monitors in use

We can't advise, comment on, or recommend specific monitors as we don't carry out any testing or research on products. The information below is intended to provide information about the most common types of monitors that are used by Trusts and Local Authorities around England, Wales and Northern Ireland.

The individual manufacturer's instruction booklet/manual should be referred to for specific information regarding use of each type of monitor e.g. interpreting alarm signals, technical specification information and regarding the cleaning of the monitor and disinfecting of sensors, as appropriate. As the average loan of equipment is for 6-8 months, equipment should usually be checked after each loan.

Skin contact monitors


Graseby MR10

This Neonatal Respiration monitor can monitor the respiration of infants up to the age of 18 months and will give audio visual alarms if the infant stops breathing for more than a pre-set time. When an infant is being monitored it is essential that the user remains within visual and audible range of the monitor so that the alarms can be seen and heard.

The breathing signal is picked up by a soft, foam-filled sensor (Respiration Sensor) that is attached to the infant's abdomen.

The sensor pad should be securely attached with hypoallergenic tape just below the umbilicus, slightly to the left or right. Inadequate skin contact is one of the reasons for a false alarm. Parents need to be provided with tape to use with their monitor.

During breathing, expansion of the abdominal wall causes a small amount of air to pass through the sensor and tube to the monitor. This triggers an audible click and simultaneous visual indication. The monitor is powered by (AA) alkaline batteries. The monitor does not require any routine maintenance apart from replacing the batteries and occasional cleaning. It is recommended that the performance of the monitor is checked at least annually. If the monitor is damaged in any way the performance must always be checked before it is used again. Sensors are single use items. They must not be cleaned or sterilized and with regular use should be replaced after a maximum of seven days. Never use the same Sensor with more than one infant. Remind parents to return any spare sensors with their monitor at the end of their time using CONI. It is important to check the expiry date, as expired sensors can give false positive alarms.

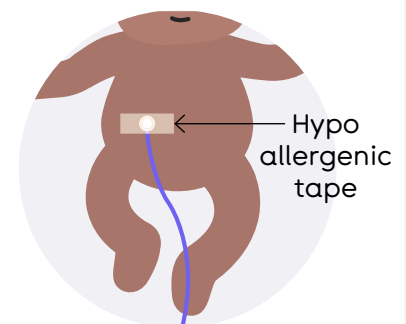


Siss Baby Control

The SISS Baby control monitors respiration in babies and infants and issues an alarm as soon as a dangerously long interruption to respiration occurs.

Respiration movements (diaphragm movements) are transformed into electrical signals in the respiration sensor via compression / bending moments.

The sensor sensitivity is largely determined by its location (next to the umbilicus) and the strength of the diaphragm movement in that position.

Fix the respiratory sensor firmly on the skin in the abdominal area using hypoallergenic tape given to parents with their monitor. Inadequate skin contact is one of the reasons for a false alarm. Sensitive and inflamed areas of the skin should be avoided. Therefore, the use of the device may be limited for babies with sensitive skin.

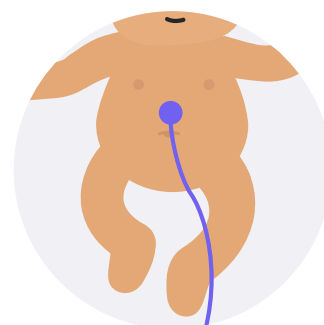

The monitor is not suitable for:

- Monitoring premature babies with a current weight of less than 1,500 grams
- Monitoring of babies and infants who do not (yet) have independent breathing
- Monitoring of intubated babies and infants
- Postoperative monitoring of babies and infants
- Monitoring premature babies or babies in incubators when the electrical components of the incubator interfere with the functioning of the device.

The sensor is reusable and should be disinfected between each baby's using the monitor. To disinfect the reusable sensor – wipe/spray the sensor with isopropyl alcohol following the disinfectant manufacturer's guidelines for relevant disinfection timescales. Then, wipe over with a soft water-moistened cloth prior to use. Do not autoclave or immerse the device or sensor in liquid. Remind parents to return the sensor with their SISS monitor at the end of their time using CONI.

Clean the surface of the device with a soft cloth moistened with water or a mild soap solution. If disinfecting is necessary, wipe the surfaces of the device with isopropyl alcohol (e.g. Clinell solution/wipes), then wipe over with a soft water -moistened cloth. Do not allow any liquid to enter any of the device's openings.

The monitor is powered by a single 9 V block battery. Siss Baby Control monitors do not require a regular annual check. However, the manufacturer recommends the following:

- Function test of monitors and accessories with every change of patient (referred to as a minor service)
- Monitor testing including possible updating of the internal software after two calendar years (referred to as a major service). Testing rigs need also to be purchased in order for servicing to take place at local level.

The sensor has a one-year warranty so is covered for this period for faults that are not due to misuse. The sensor if looked after well will continue to work. To avoid breaking the sensor advise parents not to twist but pull the sensor out from alarm box. The expected average life time is about two years.

The distribution company have produced a short user guide available for all to use via YouTube: https://www.youtube.com/watch?v=yrK_Ur461vs Parents are advised to read and follow manufacturer's instruction leaflet included in the box.

N.B. Monitors purchased prior to May 2019 are fitted with a 3-pole socket. Thereafter, they have a 4-pole socket, so it is essential that the sensor being used has the same corresponding number of poles i.e. 3 or 4 so that they can be connected together.

Snuza Hero^{MD}

The Snuza Hero^{MD} is a portable baby movement monitor which monitors a baby's abdominal (breathing) movements while sleeping. The Hero can be clipped onto the baby's nappy. No cords, wires, sensor pads or external power are required. The Hero detects even the slightest abdominal movement and will alert if the baby's abdominal movements are very weak or fall to less than eight movements per minute. If no abdominal movement at all is detected for a period of 15 seconds, the Hero will vibrate gently. After three vibration/rouse incidents, the Rouse Warning will alert to the fact that the baby's abdominal movements have stopped for 15 seconds on three occasions. If no further abdominal movement is detected for another five seconds, an alarm will sound. The monitor is powered by a battery.



Clean by gently wiping with a damp cloth to remove any dirt. A mild soap may be used for stubborn dirt. Do not use any other cleaning agents as

they may contain damaging chemicals. The monitor is splash resistant, not waterproof. Do not immerse in water. If it gets wet, dry it off immediately.

The Snuza Hero^{MD} monitor is a single use monitor and so cannot be re-loaned to another family. Parents are advised to read and follow manufacturer's instruction leaflet included in the box and watch the video of the basic features of the Snuza HeroMD on the Snuza website: [snuza.com/product/hero-md](https://www.snuza.com/product/hero-md/) <https://www.snuza.com/product/hero-md/>

Mattress pad monitors

Angelcare AC25-1

Angelcare's new medically certified AC25-1 Baby Movement Monitor is a mattress type monitor with video and sound. It uses a wireless movement sensor pad with four sensitivity levels. An alarm will sound on the Parent Unit if after 20 seconds movements are not detected by the Sensor Pad. The monitor is effective up to 150 metres and includes a nightlight and lullaby feature. It also has two-way-talk-back, room temperature display and a large 4.3" LCD screen with push button operation. A video screen does not replace in-person supervision of a baby. The Angelcare monitor is a single use monitor and so cannot be re-loaned to another family.



There is an instruction video on Angelcare's website: <https://youtu.be/-2YjxT2HC8s> and a support and FAQs section: angelcarebaby.com/en-uk/pages/support-faqs

Parents are advised to read and follow manufacturer's instruction leaflet included in the box.

Nanny BM-02 and BM-03 Baby Breath Monitors

The Nanny monitor is certified medical devices that monitors a baby's breathing movements. Their purpose is to give a visual and sound warning of any decrease in breathing movements or long pauses in breathing movements (apnoea).

The device consists of a sensor pad containing a sensitive movement detector that is placed under the baby's mattress and an alarm unit. It is battery operated and runs an auto-function test upon start up. It does not affect or restrict that baby's movements. The Nanny monitor is suitable for babies weighing from 2kg up to 15kg. The Monitor

is powered by (AA) alkaline batteries.


Parents are advised to read and follow manufacturer's instruction leaflet included in the box and watch the instruction video on the website: [How Nanny Works | Reliable Nanny Breath Monitor](#)



Other than battery replacement, the Nanny monitor does not require any special maintenance. It is recommended that the sensor pad in the cot is kept flat. It should be checked regularly for breaks in the plastic cover and occasionally checked to make sure that no moisture is present where it touches the mattress. The pad should be turned 180° occasionally and aired.

The sensor pad should be cleaned with a slightly moist cloth. (Do not use aggressive detergents). Contact with water can cause damage to the Nanny. Use antibacterial wipes for disinfecting the pad, as necessary. It is recommended to replace skin sensor monitors that have trailing wires with a mattress monitor for babies who need to be monitored after six months.

The pad is single use when used in the a domestic setting (not hospital) so every baby that is loaned the alarm unit needs a new sensor pad. The pad may be recycled at a recycling centre or in a red container for recycling small electrical waste. Some councils will collect small electrical items as part of recycling collections. The manufacturer has guidance: monitornanny.com/faq

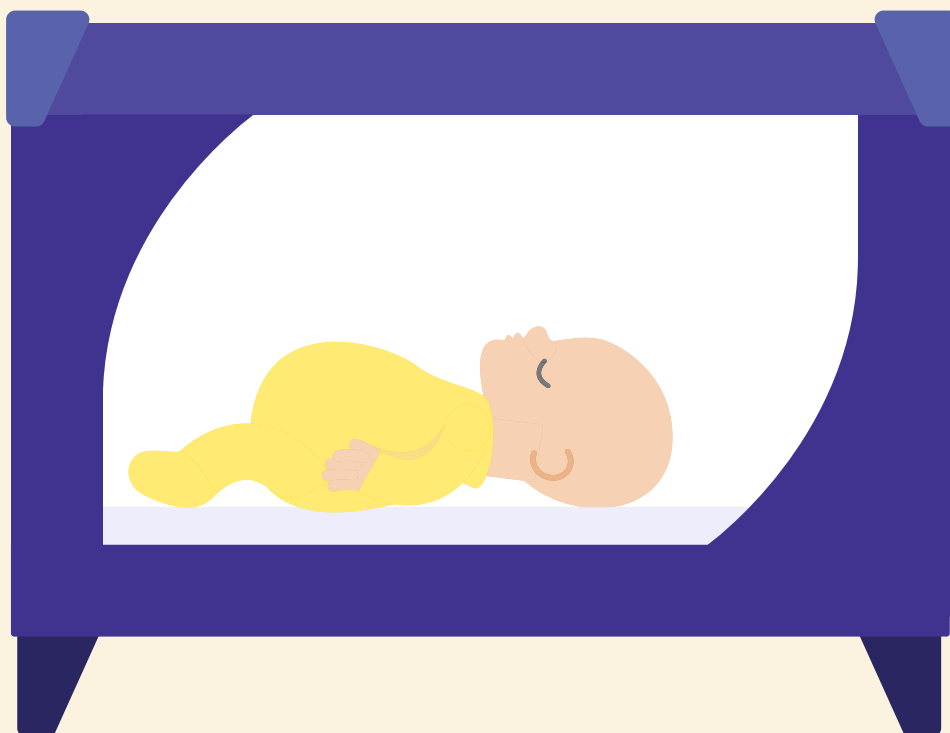

References

1. Fleming PJ, Blair P, Bacon C, Berry J. Sudden Unexpected Deaths in Infancy: the CESDI SUDI Studies. 1993-6. 2000, The Stationary Office, London.
2. SIDS and Other Sleep-Related Infant Deaths: Updated 2016 Recommendations for a Safe Infant Sleeping Environment
SIDS and Other Sleep-Related Infant Deaths: Updated 2016 TASK FORCE ON SUDDEN INFANT DEATH SYNDROME Pediatrics; 2016; DOI: 10.1542/peds.2016-2938
3. Strehle E-M, Gray W, Gopiseti S et al. Can home monitoring reduce mortality in infants at increased risk of sudden infant death syndrome? A systematic review. *Acta Paediatrica* 2012 Jan;101(1):8-13.
4. Samuels MP, Stebbens VA et al (1993). Deaths on infant apnoea monitors. *Maternal and Child Health* September 1993: 262-266
5. Sherratt SJ (1999). The pros and cons of movement monitors, *British Journal of Midwifery*, 7: 569-572
6. Waite AJ, McKenzie A, Daman-Willems C. CONI: confirmation of continuing relevance after 20 years. *Community Practitioner* 2011; 84:25-29.7. Milner AD (1985). Apnoea monitors and sudden infant death. A report from the Foundation for the Study of Infant Deaths and the British Paediatric Respiratory Group. *Arc Dis Child* 60: 76-80
8. Emery JL, Taylor EM, Carpenter RG, Waite AJ Apnoea monitors and accidental strangulation(letter) 1992 *BMJ* 304:117
9. Ward SL, Keens TG et al (1986). SIDS in infants evaluated by apnoea programs in California. *Pediatrics* 77: 451-458.



**For every baby.
For every family.
Forever.**

[lullabytrust.org.uk](https://www.lullabytrust.org.uk)

T: 020 7802 3200

Information line: 0808 802 6869 (Freephone)

Registered charity number: 262191